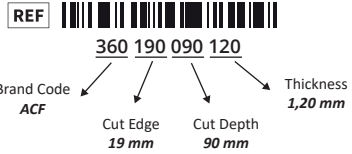
# INSTRUCTIONS FOR USE

## Sterile Surgical Saw Blade

•Surgical Saw Blades are used by specialist surgeons, technicians in the field of orthopedic and traumatology for shortening time of cutting and revision of operational process of large bones (Femur, Tibia and Humerus) in orthopedic surgery. Blades can be used for the functional control by bioengineers before the operation.

 Caution, consult accompanying documents	 Do not use if package is damaged
 Consult instructions for use	<b>REF</b> Catalog number
<b>STERILE</b>  Method of sterilization: irradiation	<b>LOT</b> Batch code
 Use by date	 Date of manufacture
 Do not reuse	<b>QTY</b> Quantity
 Temperature Limits	 Humidity Limits
 Manufacturer	<b>CE</b> European Conformance

Surgical Saw Blades are used as follows;

### WARNINGS

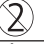
- W1. Reusing is not allowed. The operator is responsible for any risk that may arise from the saw blades used for the second time.
- W2. The principles of aseptic technic should be followed while unpacking the sterile surgical saw blade.
- W3. The sterile surgical saw blade package should not be opened before the moment it is going to be used.
- W4. It should be used in the operating room environment where appropriate conditions are provided for orthopedic operations.
- W5. If the patient shows signs of irritation and / or sensitization, the operation should be stopped.
- W6. Do not use if the sterilization label is not red.
- W7. The blades shouldn't be used if the package is damaged because they do not preserve their sterility. These blades should be returned to the manufacturer.
- W8. It is recommended to store between 13°C - 26°C degrees and %19 - %54 humidity rates.
- W9. All staff (doctors, assistants, etc.) who will install and use the product should be familiar with surgical saw blades and the related operation.
- W10. Do not exceed the manufacturer's recommended operating limits for the tool.
- W11. Ensure that you have chosen an appropriate saw blade for your cutting guide.
- W12. Monitor the temperature of the saw blade periodically. Using irrigation while operating blades will help reduce the possibility of thermal necrosis.
- W13. Ensure that the saw blade is properly inserted and securely locked before activating the tool.
- W14. The blades must be able to move freely within the guide.
- W15. Use care when handling sharps to prevent accidental cutting.
- W16. It is recommended to have a spare blade during operation.
- W17. Forceful some other side rather than oscillation may cause the fracture of the saw blade.
- W18. A blade with a bent axe will wobble and should not be used.
- W19. Eye protection should be worn at all times.
- W20. If you notice any visible dirtiness, crack or trace on the saw blades, do not use and return them to to manufacturer.
- W21. Ensure that the used blade is disposed according to the legal medical waste regulations referred to in international agreements.
- W22. The sterile surgical saw blade should be used before the expiry date declared on the label.

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